

CERVICAL COLLAR Recommendations

The cervical collar is an orthotic used for the neck to significantly reduce movements and weight of the head. This is a comfortable device that allows the skin to breathe. The height of the chin can vary to ensure a custom fit.

Installation

1) Place the front part under the chin, ensuring that it is well centered. The chin must be firmly seated on the chin rest, looking straight ahead.

2) Place the posterior part by fixing the straps of velcro on each side. It is important to position the straps symmetrically to center the posterior tie. The collar should be tight but comfortable. Be careful to position the posterior part where the writing on the straps are in place.

Entretien

When delivering your collar, a set of spare cushions were also provided. We recommend alternating the cushions each day for proper hygiene. You can wash them by hand and let them air-dry.



Adjustment wheel

- The chin must be parallel to the ground.
- During the meal, you can lower the chin-rest slightly to facilitate chewing. Do not forget to go back later.

At any time, you can contact your orthofist for adjustments to ensure your comfort. Do not hesitate to contact us.

Sherbrooke, East
360, Galt East Street, # 320,
Sherbrooke (Québec) J1G 1X9
T: 819 340-1234
info@laboratoireevo.com

Granby
459, Principale Street,
Granby (Québec) J2G 2W9
T: 450 777-3221
info@laboratoireevo.com

Evo Orthopédie Technique
1851, Sherbrooke E Street, #401,
Montréal (Québec) H2K 4L5
T: 438 386-6966
montreal@laboratoireevo.com

Sherbrooke, West
21, boul. J.-A.-Bombardier
Sherbrooke (Québec) J1L 0H8
T: 819 340-1234
info@laboratoireevo.com

Lac-Mégantic
6250, Salaberry Street,
Lac Mégantic (Québec) G6B 1H8
T: 819 340-1234
info@laboratoireevo.com