

JEWITT CORSET Recommendations

The Jewitt corset must be installed with care on the patient. It is essential to follow the instructions given by the orthotist to ensure its effectiveness. This corset reduces the compression between the vertebrae of the spine to reduce pain and promote healing. It is often used during a fracture thoracic vertebrae.

Installation

- 1) Always wear a tight sweater under the corset.
- 2) Make sure the loop is open and that the fastener on the other side is detached from the rigid frame.
- 3) Lying on the side, rolling onto your back by placing the cushion dorsal in the middle of the back. Place the rigid frame on your torso. With the help of the thumb strap, attach the loop on the rigid frame.

Close the tension loop on the other side, making sure to hear the clicking sound.

Maintenance

The rigid frame can be cleaned with a mild soap and hot water. You can also clean it with a mixture of water and alcohol.

The cushions can be removed and cleaned with a mild soap and warm water. It can then be flat-dried.



At any time, you can contact your orthotist for adjustments to ensure your comfort. Do not hesitate to contact us.

Sherbrooke, East
360, Galt East Street, # 320,
Sherbrooke (Québec) J1G 1X9
T: 819 340-1234
info@laboratoireevo.com

Granby
459, Principale Street,
Granby (Québec) J2G 2W9
T: 450 777-3221
info@laboratoireevo.com

Evo Orthopédie Technique
1851, Sherbrooke E Street, #401,
Montréal (Québec) H2K 4L5
T: 438 386-6966
montreal@laboratoireevo.com

Sherbrooke, West
21, boul. J.-A.-Bombardier
Sherbrooke (Québec) J1L 0H8
T: 819 340-1234
info@laboratoireevo.com

Lac-Mégantic
6250, Salaberry Street,
Lac Mégantic (Québec) G6B 1H8
T: 819 340-1234
info@laboratoireevo.com